



## Feet - Diabetic Plantar Ulcer



**Plantar Ulcer**

This plantar ulcer had been non-healing for an extended period of time. It was determined to use DermaClose to accelerate the time-to-closure.



**post-debridement**

After extensive debridement the wound was ellipsed thereby converting this wound to an acute wound.



**Post-application**

The DermaClose was positioned remotely utilizing the bridge tubing and tensioned. The wound was dressed with an absorbent dressing to manage wound



**Day 4**

On day four the wound was fully approximated, the DermaClose removed and the wound sutured closed without complication.

**DERMA**Close<sup>®</sup> RC  
Continuous External Tissue Expander

**WOUND**CARE<sup>™</sup>  
TECHNOLOGIES  
"Expanding Your Clinical Options"

Expanding your clinical options

www.dermaclose.com • (800)896-0436